





# BD Stent

SX-ELLA Stent Esophageal Degradable BD

**Worldwide unique self-expandable  
esophageal biodegradable stent**

For management of refractory  
benign esophageal strictures

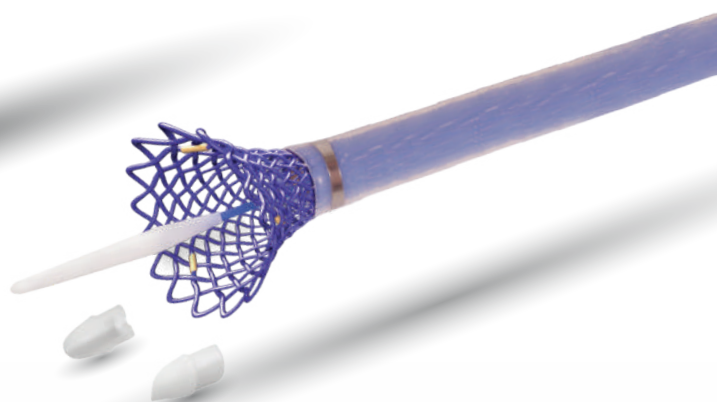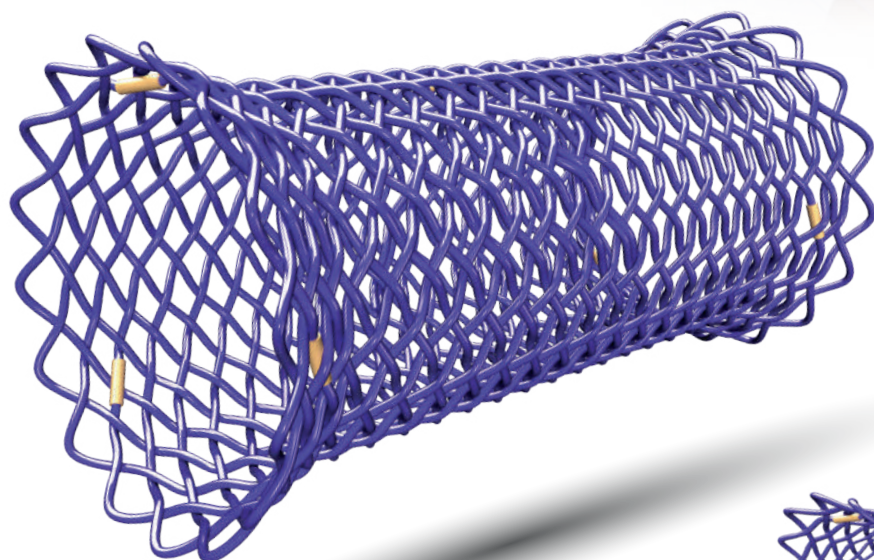

Designed & manufactured  
in the Czech Republic (EU)

[www.ellacs.eu](http://www.ellacs.eu)

3D ANIMATION



Manufacturer:  
**ELLA-CS, s.r.o.**  
Milady Horakove 504/45, Trebes  
500 06 Hradec Kralove, Czech Republic

## Indication

BD Stent is intended for patients aged 18 and older and indicated for management of refractory benign esophageal strictures:

refractory esophageal strictures (peptic, caustic, or anastomotic) in which the standard therapy with repeated dilation failed.

## Technical information

Stent is to be compressed by compression tool just prior to implantation.

6 mm / 9.4 mm (18 F / 28 F) delivery system, available in standard length of 75 cm.


Recommended guide wire: 0.035" (0.89 mm) ultra stiff / Minimum length 220 cm.

## Features and Benefits

- + **Biodegradable material - Polydioxanone** - stent degrades usually 3 to 4 months after the implantation; no need for removal.
- + **Integrity and radial force of stent is maintained for 6–8 weeks following implantation** - extended period of dilation compared to conventional methods.
- + **Anti-migration design** - dual flared design; reduced migration rates.
- + **Radiopaque markers** - accurate stent positioning.
- + **Simple compression procedure** - compression tool available.
- + **PULL-back delivery system** - splittable olive tip - easy insertion and safe delivery system removal.

## Available sizes

| REF No.              | Nominal stent body diameter [mm] | Nominal stent flares diameter [mm] | Nominal stent length [mm] | Delivery system      |
|----------------------|----------------------------------|------------------------------------|---------------------------|----------------------|
| 019-10A-23/18/23-060 | 18                               | 23                                 | 60                        | 28 F / 18 F<br>75 cm |
| 019-10A-23/18/23-080 |                                  |                                    | 80                        |                      |
| 019-10A-23/18/23-100 |                                  |                                    | 100                       |                      |
| 019-10A-25/20/25-060 | 20                               | 25                                 | 60                        |                      |
| 019-10A-25/20/25-080 |                                  |                                    | 80                        |                      |
| 019-10A-25/20/25-100 |                                  |                                    | 100                       |                      |
| 019-10A-28/23/28-060 | 23                               | 28                                 | 60                        |                      |
| 019-10A-28/23/28-080 |                                  |                                    | 80                        |                      |
| 019-10A-28/23/28-100 |                                  |                                    | 100                       |                      |
| 019-10A-31/25/31-060 | 25                               | 31                                 | 60                        |                      |
| 019-10A-31/25/31-080 |                                  |                                    | 80                        |                      |
| 019-10A-31/25/31-100 |                                  |                                    | 100                       |                      |
| 019-10A-31/25/31-135 |                                  |                                    | 135                       |                      |

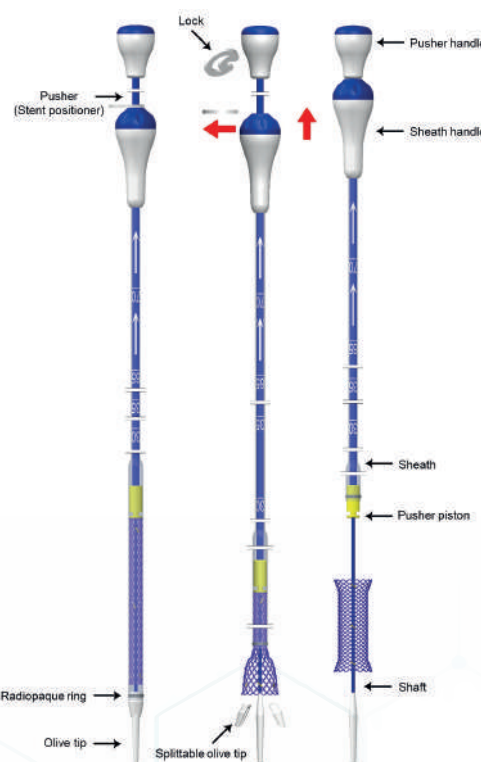

## PUBLICATIONS



[www.ellacs.eu](http://www.ellacs.eu)

[info@ellacs.eu](mailto:info@ellacs.eu)



Compression tool (included in the packaging)

CE 1014

Manufacturer:  
**ELLA-CS, s.r.o.**  
Milady Horakove 504/45, Trebes  
500 06 Hradec Kralove, Czech Republic

SX-10A-AJ-10/10/P/REV-10-04/2024